
Instructions for use
Mode d'emploi
Gebrauchsanleitung
Instrucciones de uso
Istruzioni per l'uso
使用说明

Amielle[®]
Care

4 Graduated sizes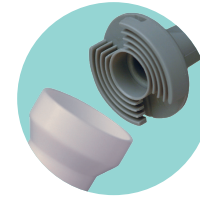

Twist & lock handle

As a result of your recent treatment, your vagina may have become narrowed and shortened due to the formation of scar tissue.

Amielle® Care vaginal dilators have been designed to greatly reduce your risk of scarring and assist you in keeping your vagina supple. This will help reduce any long term discomfort from follow up examinations, further treatments and sexual intercourse.

Amielle® Care restores vaginal capacity and elasticity and can help to alleviate sexual discomfort.

This guide was written in conjunction with The UK National Forum of Gynaecological Oncology Nurses.

Please read these instructions fully before using Amielle® Care vaginal dilators for the first time.

How to use your Amielle® Care vaginal dilators

Before You Begin – Prior to first-time use, wash dilators and handle in warm, soapy water, rinse thoroughly and allow to dry.

Privacy – Use the dilators in a private, comfortable setting where you will be undisturbed.

Positioning – The recommended position to use the dilators is to lie flat on your back with your knees bent and legs slightly apart. Alternatively, you may stand with one leg raised on a chair.

Progression – Start with the smallest dilator and, once you can comfortably insert it, progress to the next size up. You may find when you first use the dilators that you can only tolerate the tip of the dilator. With persistent use you will be able to gradually progress to full insertion, then larger sizes. When transitioning sizes, move from the smaller dilator to the next size in the same session, as vaginal muscles will already be relaxed and the transition will be easier. The time it takes to progress from size to size varies by individual.

Duration & Frequency – Duration and frequency of dilation may range from three minutes twice a week up to ten minutes and twice daily*. Please consult with your Healthcare Professional.




Cleaning Instructions – Amielle® Care vaginal dilators and handle, should be washed after each use. Wash in warm, soapy water, rinse thoroughly and allow to dry before storing in carrying bag. Ensure all traces of soap are removed to avoid irritation. Remember to wash the dilator handle if it has been used.

Warnings and Precautions

These instructions are intended as a guide and should be used in conjunction with the advice given by your Healthcare Professional.

If any serious incident occurs with the use of this device please report to Owen Mumford Limited and your local Competent Authority.

* International Guidelines on Vaginal Dilation after Pelvic Radiotherapy, June 2012.

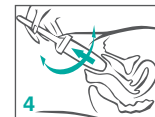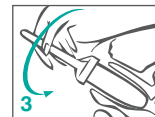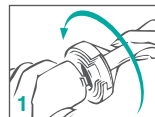Inserting your dilator

1. Twist and lock handle onto dilator. Apply lubricating jelly to the tip of the dilator and to your vaginal opening.
2. Relax as best as possible. Using controlled breathing, slowly ease the dilator into the vagina going as deeply as is comfortable. Since the vagina slants upwards and towards your back, the dilator should be angled upwards and in the direction of your back during insertion.
3. Once the dilator is inside the vagina move the dilator in a forward and backward motion, then a left to right motion and if possible, gently rotate the dilator in position by twisting the handle.
4. Remove gently and slowly while rotating.

For demonstration video visit: <https://www.owenmumford.com/en/patients-product/amielle/>. Find Amielle® on YouTube: <https://www.youtube.com/watch?v=ljuW3qL9sE0> or search for Amielle® Care.

Warranty Information

The warranty offered by Owen Mumford Limited (“we”) in relation to this product is set out at [owenmumford.com](https://www.owenmumford.com). PLEASE READ THIS WARRANTY IN FULL. Your rights under the laws of your country are not affected.

Depending on the product you purchased, your pack may contain a lubricating jelly. If the pack contains a lubricating jelly you will have received OptiLube. See below for more information on this lubricating jelly. Amielle® Care can also be used with Aquagel® or Sylk which is not supplied with the pack.

OptiLube Lubricating Jelly

Ingredients: Purified water, Polyethylene glycol, Glycerine, Carbomer, 4 hydroxy benzoic acid esters.

Keep dry, keep away from direct sunlight.

Made by Optimum Medical Solutions Ltd, Tennant Hall, Blenheim Grove, Leeds, LS2 9ET, UK. www.optimummedical.co.uk

2797

STERILE R

OptiLube, Aquagel® and Sylk and are not products or trademarks owned by Owen Mumford. For more information please contact the relevant manufacturer.

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Owen Mumford GmbH,
Freyr Life Sciences GmbH, Bahnhofplatz,